OXYGEN

FINGER PULSE OXIMETER

MANUAL ECO001

OPERATING OVERVIEW

- Check the device to make sure there is no visible damage that may affect the user's safety or measurement performance with regard to sensors and clips. It is recommended that the device should be inspected minimally once a week. When there is obvious damage, stop using the device.
- Necessary technical maintenance must be performed by qualified service technicians. Users are not permitted to maintain it by themselves.
- The oximeter cannot be used together with devices not specified in this Instruction Manual.
- Explosive hazard – DO NOT use this oximeter in an environment with inflammable gas such as some ignitable anesthetic agents.
- DO NOT use the oximeter while the testee is under MRI or CT scanning.
- An uncomfortable or painful feeling may appear if using the oximeter continuously on the same place for a long time, especially for poor micro circulation patients. It is recommended that the oximeter should not be applied to the same location for longer than two hours. If any abnormal condition is found, please change the position of oximeter.
- DO NOT clip this device on edema or tender tissue.
- The light (infrared light is invisible) emitted from the device is harmful to the eyes, so service technician or testee should not stare at the light.

- The local law must be followed when disposing of the device.
- Keep the oximeter away from dust, vibration, corrosive substances, explosive materials, high temperature and moisture.
- The device should be kept out of the reach of children.
- If the oximeter gets wet, please stop using it and do not resume operation until it is dry. When it is carried from a cold environment to a warm and humid environment, please do not use it immediately.
- DO NOT operate the button on the front panel with sharp materials.
- DO NOT use high temperature or high pressure steam to disinfect the oximeter.
- DO NOT use it when the sensor is damaged.

BATTERY INSTALLATION

1. Slide cover off from backside of pulse oximeter.
2. Insert two "AAA" batteries into the compartment, matching the indicated polarity symbols.
3. To close, slide cover back into place. It is recommended to remove the batteries if the unit will not be used for an extended period of time.

IMPORTANT INFORMATION BEFORE USE

- The finger should be put in properly and correctly.
- Do not shake the finger. Keep at ease during measurement.
- Avoid placing the device on the same limb which is wrapped with a cuff for blood pressure measurement during venous infusion.

- Do not let anything block the emitting light from device.
- Vigorous exercise and electro-surgical device interference may affect the measuring accuracy.
- Using enamel or other makeup on the nail may affect the accuracy of the measurement.

WHAT IS PULSE OXIMETER

The Pulse Oximeter, based on all digital technology, is intended for noninvasive spot-check measurement of functional oxygen saturation of arterial hemoglobin (SpO₂). Advanced DSP algorithm (Digital signal processor algorithm.) can minimize the influence of motion artifact and improve measurement accuracy of low perfusion (In physiology, perfusion is the process of a body delivering blood to a capillary bed in its biological tissue. Under the condition of low perfusion, the measurement of non-invasive saturation of pulse-blood oxygen is low-accurate.).

The Oximeter can be used to measure human Hemoglobin Saturation and heart rate through finger. The product is not only suitable for use in family, hospital (including clinical use in internist/surgery, Anesthesia, pediatrics, intensive care and etc.) and also in the oxygen Bar, social medical organizations, physical care in sports and etc.

PART DESCRIPTION

- (1) Cushions
- (2) ON button
- (3) Display screen
- (4) Hanging hole

USE THE DEVICE

1 Open the clip and place finger into the cushions of the clip, and release the clip.

2 Press the ON button to begin measurement.

3 The device starts measurement, you will hear the beep sound. After 3 seconds, your measurement result will be show on the display.

4 Remove finger and the oximeter will power off automatically.

DESCRIPTION OF DISPLAY SYMBOLS

%Spo2 Pulse oxygen saturation

♥/min Pulse rate (beats per minute)

Low battery indicator 

CLEANING

• Clean with a soft cloth by wetting with a solution, such as 75% isopropyl alcohol. If low-level disinfection is required, use a 1:10 bleach solution.

Caution:

Do not sterilize by irradiation steam or ethylene oxide.

MAINTENANCE

In order to ensure long service life for your pulse oximeter, please follow the below maintenance guidelines.

- Change the batteries when the low indicator light appears.
- Clean the surface of the device before using.

Caution:

- Do not immerse device in liquid.
- It is recommended that the device should be kept in a dry

environment. Humidity may reduce the life of the device, or even damage it.

TROUBLESHOOTING

The SpO2 and pulse rate are unstable.	Make sure the finger is inserted correctly. Review the “Important Information Before Use” section.
The device will not turn on.	Check and correct the polarity of the installed batteries. Reinstall or replace batteries.
Display is blank.	Check and correct the polarity of the installed batteries. Reinstall or replace batteries.

SPECIFICATIONS

Power supply: 2 *AAA batteries(1.5V)

Supply voltage: 3V

Operating current: $\leq 40\text{mA}$

SpO2 parameter: Measuring range: 35% - 99%

Accuracy: 70% - 100% $\pm 3\%$

Pulse rate: Measuring range: 30bpm - 240 bpm

Parameter accuracy: $\pm 2\text{bpm}$ or $\pm 2\%$

Operation temperature: 41°F - 104°F (5°C - 40°C)

Environment humidity: 30% - 80%

Pressure: 70kPa - 106kPa

Performance under low perfusion condition : Same with modulation amplitude as low as 0.2%

Resistance to light interference: $\pm 1\%$

Resistance to 50Hz/60Hz interference: Precise

WARRANTY

1. This warranty is issued by Esperanza sp.j, Poterek with address:
ul. Poznańska 129/133, 05-850 Ożarów Mazowiecki, Phone: 022-7213571 w.10 – called hereinafter Esperanza.
2. This warranty confers upon you specific rights and remedies against Esperanza. Please read this document carefully, by making a claim under this warranty, you will be deemed to have understood and accept its terms.
3. Esperanza warrants that any Esperanza product together with attached accessories is free from defects in materials and workmanship under conditions of its normal use.
4. If you make a claim under this warranty, Esperanza will, at its option, repair or replace any defective product into new one by using new or use factory-refurbished parts. In case of using factory-refurbished parts they will be equal to new products.
5. Defective product should be returned together with receipt, commercial invoice or any other proof of purchase to the place of purchase. It should include all accessories attached in original packing. Esperanza has right to reject warranty if any accessories will be missed.
6. Esperanza is committed to respond to the complaint within 14 calendar days of its notification in the point of sale. Esperanza will repair the defective product in the shortest possible time. If will be need to import from abroad of any parts which are necessary to repair this time may be extended.
7. In case of replacing defective product onto new one, Esperanza has right to replace it with newer model with the same technical parameters if the old one model was replaced by new one during warranty period.
8. If you make claim under this warranty, you are deemed to have agreed that repair and (upon availability) replacement, as applicable, under the warranty services described herein, is your sole and exclusive remedy vis-a-vis Esperanza.

EXCLUSIONS:
This warranty does not extend to:

1. Any product not manufactured by Esperanza
2. Any product that has been damaged or rendered defective as a result of:
 - use it in other way than normal intended use
 - modification or internal interferes or changes of product
 - service or repair by anyone other than Esperanza service
 - improper store, transportation or packing
 - improper installation of product or software
 - mechanical, physical, chemical etc. damages
3. In no event shall Esperanza be liable for any damages, loss of business, profits, data or loss based on use Esperanza products with other devices.
4. Products with removed warranty and other labels and products spent or have hard use marks.

All products are covered by Esperanza's 2-year manufacturer's warranty while product is purchased by the consumer (the consumer is a person who buys product for the purpose of not professional or commercial activity). In the case of purchase for the company (for purposes relating to the activities of professional or commercial) warranty is 1 year from date of purchase. Esperanza products have all the necessary safety certificates required in the EU.

According to Directive 2002/96/EC, disposal of waste electrical and electronic equipment together with household waste is prohibited. Their components need to be recycled or disposed separately. Otherwise, they can cause the release of hazardous and noxious substances which pose a threat to the environment. The law requires each consumer to return unnecessary or waste electrical and electronic equipment to the producers, sales agents or waste treatment facilities. Detailed conditions are governed by national legislation. The above symbol placed on the product, in the user manual or on the packaging refers to this requirement. Thanks to proper segregation and compliance with the provisions relating to waste disposal each user contributes significantly to environmental protection.

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We, the manufacturer/importer (My, producent/importer):

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Is in compliance with the essential requirements and other relevant provisions of the following directives:

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EMC DIRECTIVE 2004/108/EC
Year of issue (last 2 digits): 15
MEDICAL ELECTRICAL EQUIPMENT

The products is compatible with the following norms/standards:

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Manufacturers/Authorised representative

Product Manager
Ozarów Mazowiecki 31-01-2016

Producent/ Produçador/ Producer/ Výrobca/ Výrobca
Produtor/ Gamintojas/ Produtor/ Producteur/ Gyártó
Produttore/ Produzent/ Παραγωγός/ Производителъ:
EDC Sp. Jawna. 05-850 Ozarów Mazowiecki
ul. Poznańska 129/133 www.esperanza.pl
MADE IN P.R.C.